ラブテックパソコン心電計 Duna

EC-12RS 簡易取り説

- ☆ ストレステスト記録
- 1. 画面の立ち上げ アイコンを左ダブルクリックしてください
 2. モードを安静時 からストレステスト ふ切り替えてください
 3. 被験者情報入力 ジェタンを押してください

an and a go statistics.		1	New Patient
First Name	Last Name	Patient ID Last Record Dat	A
aare	Good	12345	Modify
mergency	0001	Emergenc 2007-08-08	
recometer	Patient 4	15704301 2006-10-12	Delete Patient
rgometer	Patient 5	017-879 2006-03-30	Citra hu
reometer	Patient_6	015-887 2006-03-30	Filler by.
rgometer lacter	Patient_/	1 2006-10-15	[Any ECG Type]
eiichiroh	Fukuyama	777 N 2007-07-14	[Any Time]
est	1	111111 2007-07-18	i i i i i i i i i i i i i i i i i i i
est	2 Patient1	222222 2007-01-24	[Any Physician]
est	Patient2	097957733 2007-07-15	[Any Institute]
est	Patient3	053120105 2003-06-24	- I I I I I I I I I I I I I I I I I I I
mergency Patien	£		
ords:			
ords: ype D	ate	Physician Filename	New Record
ords: ype D	ate	Physician Filename	New Record
ype D	ate	Physician Filename	New Record
ords:D	ate	Physician Filename	New Record
ords:D	ate	Physician Filename	New Record
ords:	ate	Physician Filename	New Record
ords. ype D	ate HDD / Actual Data	Physician Filename	New Record
ype D	HDD / Actual Data	Physician Filename	Load Record Delete Record Records)
ype D	ate HDD / Actual Data Ac Free Space in	Physician Filename base Location: "TSUKAHARA-10BECG tual Database: 164.3 MByte (15 Patients, 81 This Directory: 3806.1 MByte (76.1 Records)	Load Record Delete Record Records)

VH	17	
ťK	K_	

ボタンを押して各項目を入力してください

New Patient		
Patient	First Name	Last Name
	Date Of Birth 1977-08-14	Patient ID
Address		
Height Weight	cm kg	Sex
	ОК	Cancel

New Patient

- ボタンを押して Default Stress Device から負荷モードを選択
- してください

4. セッティング

Nuedar of New Records Integratation	:
Network Chross Device Betting Stress Device and Protocol. IT Calmilation Ressure of ECG.	Ergumeter Treadmill
Select Colors. Select Physician. Search Ecg Devices. Lead off memory.	67
Advanguage Translator. Janvics. Jahug.	Ů

- Start .oad 5. スタート ボタンを押してストレステストを開始してください
- 6. プロトコル終了後自動印刷されます

自動記録が設定されていないときはプリント ボタンを押してください

Print Preview			_ 8 ×
Selected Pages Page title Pages Copies Prev	view Page title Pages Copies Preview Sele	ected Lead V5 💌	
Not Implemented Yet > 0 1 100	Not implemented Yet 0 1 . C Sta	indard (I - III)	
ECG Curves)1 1	Not Implemented Yet 0 1	ipolar (aVF - aVR)	
Not Implemented Yet > 0 1	Not Implemented Yet 0 0 120 C Sta	indard + Unipolar	
🗆 Not Implemented Yet 🛛 🙀 🔳 🕐	Not implemented Yet 0 1 MC C 12	Leads	
Page: 1 / 1 Zoom: Fr	t Width Total pages: 1 Print	Cancel	
Selected Printer:	Color Mode Printin	ig Mode	
Canon iP6600D	Setup C Black and White G Full Color	C Automatic C Manual	
			_
Physician: Test Physician	Recorded: 14-07-2007, 14:11 h. Exercise time: 00:00 min. Load: 0W	Institute: XZ	Page: 1/1
Patient: Seiichiroh Fukuyama	HR: 0 b/min	Interpretation of Rest ECG:	
Age: 42	Intervals (msec): Pd: 0 PO: 0	ECG curves not usable.	
Patient ID: 777 Rem: 21.10.1984 Rem: Male	QRS: 0 QT: 0 QTc: 0		
Height cm Weight 75 kg	Axis (degree):		
Comment:	r. • • • • • • • • • • •	-	
		그 아무 아무 가지 가지 그 아무 가 가 가 가 아니는 거	
		1	
╎╴┙╎┧┟┚┝╍┙╲┧┍┙╵			A 7
		s den den den ste ste nie om met met met met met met met met ste ste ste ste ste ste ste ste ste s	
	ni in la Muinte de la		
 A second s	The second	The second s	a second s

次に

Print

ボタンをおしてください